



Let's get you started with InPen.

Patient Name

Today's Date

Prescriber

To learn more visit www.companionmedical.com or call **844.843.7903**

Welcome

InPen™ is the first and only *Smart Insulin Pen + App* system that gives you the tools to manage your multiple daily injections.

InPen is for you if:

1. You want a more user-friendly alternative to vial + syringe.
2. You want a smart, more sophisticated insulin pen that logs and calculates doses for you.
3. You want reminders for mealtime and long acting doses.
4. You need a break from your pump or want a backup to your pump when traveling.

Now, let's get you started with the future of diabetes care!

Join the community



@companionmedical #myinpen

When picking up your new InPen

Supplies Needed

InPen, insulin cartridges, pen needles, Smart Phone (iOS or Android).

Download the InPen App and create an account

Open App Store and search for InPen. Download InPen by Companion Medical App. Create login with email, Facebook, Gmail, or Twitter.



This information is required before first use of InPen.

For Prescriber:
Specify patient *Therapy Settings* for the following InPen App screens.

For Patient:
Take the recommended *Therapy Settings* and input into the InPen App at setup.

Your healthcare provider recommends:
Constant

Rapid-Acting Long-Acting Meal Times

Maximum Calculated Dose _____
U

Duration of Insulin Action _____
: _____
hh:mm

Time of Day Settings **OFF**

Target Blood Glucose _____
mg/dL

Insulin Sensitivity Factor _____
mg/dL/U

Insulin to Carb Ratio _____
g/U

Your healthcare provider recommends:
Time of Day

Rapid-Acting Long-Acting Meal Times

Maximum Calculated Dose _____
U

Duration of Insulin Action _____
: _____
hh:mm

Time of Day Settings **ON**

Time of Day	_____ : _____ AM / PM	_____ : _____ AM / PM	_____ : _____ AM / PM	_____ : _____ AM / PM
Target Blood Glucose	_____ mg/dL	_____ mg/dL	_____ mg/dL	_____ mg/dL
Insulin Sensitivity Factor	_____ mg/dL/U	_____ mg/dL/U	_____ mg/dL/U	_____ mg/dL/U
Insulin to Carb Ratio	_____ g/U	_____ g/U	_____ g/U	_____ g/U

Your healthcare provider recommends:
Long-Acting

Rapid-Acting **Long-Acting** Meal Times

Insulin Type _____

Doses per day _____

DOSE 1

Usual Amount _____
U

Reminder Time _____
: _____
hh:mm

DOSE 2

Usual Amount _____
U

Reminder Time _____
: _____
hh:mm

Connect blood glucose meter or CGM

The InPen System can be paired to Bluetooth® Blood Glucose Meters and Dexcom® Continuous Glucose Monitors. To optimize your therapy, talk to your healthcare provider about how to get a BGM or CGM.

How to connect your Blood Glucose Meter:

iOS: Use Apple Health to connect. On the **Settings** tab in the app, select **Sync Health Data**. Select **Connect to Health**. Once in Apple Health, turn **All Categories On** and select **Allow**.

Compatible devices: *AgaMatrix Jazz Wireless 2, Ascensia Contour® Next One, iHealth® Smart, Lifescan One Touch Verio® Flex, Lifescan One Touch Verio® Sync, One Drop Chrome, Roche Accu-Chek® Aviva Connect, Roche Accu-Chek® Guide*

Android: Not available

How to connect your CGM:

Compatible devices: *Dexcom® G5 / Dexcom® G6*

iOS: Use Apple Health to connect. On the Settings tab in the app, select **Sync Health Data**. Select **Connect to Health**. Once in Apple Health, turn **All Categories On** and select **Allow**.

Android: Under **Settings** select **Connected Services**. Select **Add Service** and sign in to allow data sharing.

Sharing your Insights by InPen Report

Send your Insights Report - To share your progress with your healthcare team, select **Reports** tab.

1. Tap **Reports** icon and pull down to generate new report.
2. In **Reports**, tap **Settings** in top left hand corner to select date range.
3. Tap **Share** icon (Upper Right).
4. Select Email or Fax.

Fax: _____

Email: _____

5. Frequency to send reports:

Weekly

Monthly

Before Visit

FAQ

How does InPen work?

The InPen, a reusable pen injector, was designed for insulin-dependent individuals on multiple daily injections. It can deliver up to 30 units of insulin, dialed in half-unit increments. InPen has a one-year warranty and does not require charging. The InPen App is the other half of InPen's smart diabetes management tool. Using information transmitted from the pen, the app can record insulin therapy, calculate doses, share therapy data with your doctor or family, and much more.

Can I pair more than one InPen to the app?

Yes, the InPen App can be paired with multiple InPens.


What if I have more questions?

Call our toll-free line at 844.843.7903

Email us at: support@companionmedical.com

Visit us at: www.companionmedical.com

Get to know your InPen

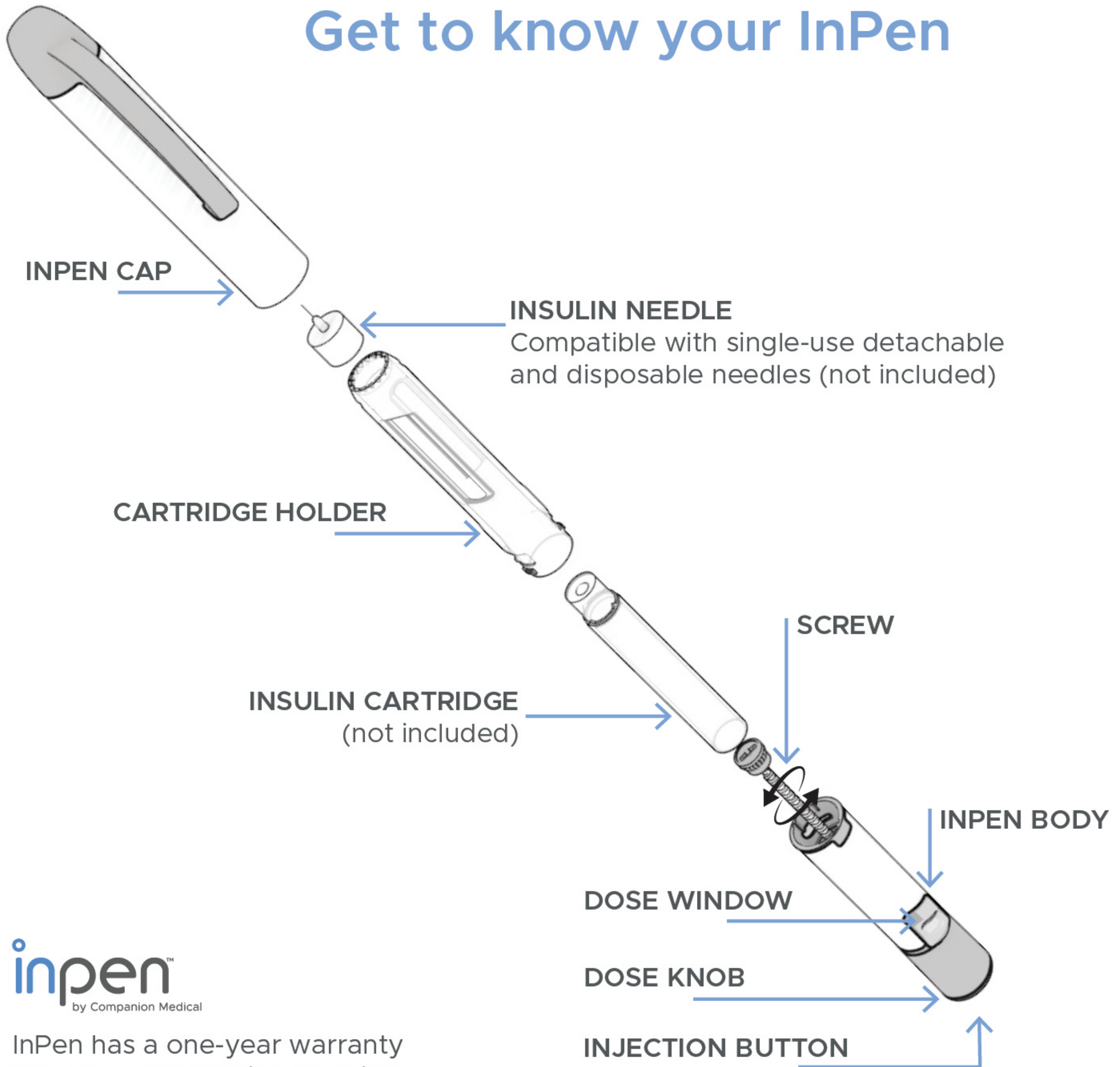

inpen[™]
by Companion Medical

InPen has a one-year warranty and does not require charging.



To learn more visit www.companionmedical.com or call 844-843-7903

*RX Only. The Bluetooth® Wordmark and logos are owned by Bluetooth® SIG, Inc. Other trademarks and tradenames are those of their respective owners. InPen logo is a trademark of Companion Medical. Copyright © 2019 Companion Medical and/or its affiliates. All rights reserved. LBL-00632-AA